

## Paper Points Directions for Use

- “Hand” rolled for extra absorbency
- Highly absorbent
- Length marked available
- Variety sizes and packages available
- Color coded

## Indication and Purpose

Paper points are used to dry the root canal before obturation and to clean the root canals that are to be sealed in endodontic treatments.

## Instructions for Use

1. Measure the length of root canal
2. The formation of the root canal of the same size as the last file to select
3. The root canal using paper points to dry
4. About 10 ~ 15 points is used per a root canal
5. Sealer or cement is applied by using paper points

## Cautions

1. Should be use sterilized paper points
2. Do not use bent or broken points
3. Should be careful to avoid overfilling or dead space when applied sealer or cement
4. Use it immediately when applied sealer or cement
5. After using the paper points, should be kept in a cool place
6. Do not expose to direct sunlight

## Storage

Store at room temperature of 1°C~30°C, away from sunlight

## Shelf Life

5 years from manufacture date

 Manufactured by **META BIOMED CO LTD**

### Osong Factory

270, Osongsaengmyeong 1-ro, Osong-eup, Heungdeok-gu, Cheongju-si, Chungcheongbuk-do, Korea

Tel: +82-43-216-0433 / Fax: +82-43-217-1988

E-mail: [info@meta-biomed.com](mailto:info@meta-biomed.com)

Web site: <http://www.meta-biomed.com>

### Cheongju Factory

136, Mochung-ro, Seowon-gu, Cheongju-si, Chungcheongbuk-do, Korea





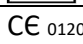



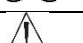
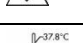
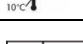
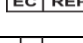




### Meta Biomed Europe GmbH

Wiesenstr. 35, 45473 Mülheim an der Ruhr, Germany

Tel: +49-208-30991910

Fax: +49-208-3099199

Symbol	Meaning (Standard, If Applicable)
	Manufacturer/Legal Manufacturer (ISO 15223-1)
	Date of manufacture (ISO 15223-1)
	Catalogue Number (ISO 15223-1)
	Batch Code (ISO 15223-1)
	CE-Mark and identification number
	Keep away from Sunlight (ISO 15223-1)
	Do Not Re-use (ISO 15223-1)
	Consult instructions for use (ISO 15223-1)
	Caution (ISO 15223-1)
	Temperature Limitations (ISO 15223-1)
	Authorized EC Rep (ISO 15223-1)
	Use By (ISO 15223-1)
	Do not re-sterilize (ISO 15223-1)
	Sterilized using irradiation (ISO 15223-1)